1 Minute UrinCheck

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First check. Then act.

1 Minute UrineCheck

EARLY DETECTION OF UTI & DIABETES
– INCL. PH TESTING

Easily test **10 parameters** in just **one minute** at home with a **99% accuracy** rate.

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10 parameter UrineCheck

General Information

The examination of urine in small animals (*dogs, cats, rabbits, guinea pigs, etc.*) can provide indications of disorders and diseases of the urine-producing and urine-excreting pathways (Urinary Tract Infections, Diabetes...). Often, the examination of spontaneous urine is sufficient to make a preliminary diagnosis.

Instructions for Use

Dip the test strip into the fresh urine for about 1 second. Wipe the side edge against the rim of the container or dab briefly on paper to remove excess urine. Compare the reaction color after 30-60 seconds (*leukocyte field after 60-120 seconds*) with the color scale. The most favorable reading time is after 30 seconds (*leukocytes after 90 seconds*). Color changes that occur after more than 2 minutes are irrelevant. The urine should not have been standing for more than 2 hours before testing.

1 Collect morning urine in a clean container

2 Immerse the test strip for 1 second and wait 60 seconds.

3 Check the result with the table provided

4 Result = positive / inconclusive. Take a photo and report it in the chat.

How to properly collect urine

To easily collect the morning urine use a clean scoop or a clean Tupperware container.

Notes

In general, individual test strip results can only enable a definitive diagnosis and specific therapy in conjunction with other medical findings.

The effect of medications or their metabolites on the test is not known in all cases.

Use only well-rinsed, clean containers for urine collection. Only the necessary number of test strips and close the package immediately after. Do not touch the reaction zones! Protect the test strips from sunlight and moisture. Store the container in a cool, dry place (4-30 °C). With proper storage, the test strips are usable until the printed expiration date.

Analysis

Place the used test strip directly next to the color scale and take a well-lit photo.

Scan the QR code below and send the image in the chat so one of our veterinarians can get back to you promptly.

Disposal

Dispose of the used test strips in accordance with applicable safety regulations.

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